



Any MAXAIR CAPR Helmet

O.N. 2070-03
O.N. 2075-03
O.N. 2081-03
O.N. 2083-03

Clear Gorilla Glue

Syringe and Syringe Tip

Symbol Definitions



Warning, Caution, or Note



Order Number



Part Number

! WARNING

Read and understand the User's Instructions Manual (UIM, P/N 03521015). Failure to follow the User's Instructions Manual may be hazardous to the user's health.

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

Prior to using any MAXAIR[®] System or component, be sure to be familiar with the system's NIOSH approved configuration.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc. does not recommend the appropriate systems for a particular institution or facility.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Use only MAXAIR Systems/ NIOSH approved compatible components.

NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen.

Follow current local regulations governing biohazard waste to safely dispose of used shrouds.

If you need more information, contact your BMDI Sales Representative, or call BMDI customer service at 1-800-443-3842.

Intended Use

These instructions are intended for gluing the front (P/N 03521039) and two side (P/N 03521038) adapters to any MAXAIR CAPR Helmet.

These adapters are compatible with CAPR Filter Cover Cap O.N. 2061-08 and CAPR Hard Hat O.N. 2061-04A, and allow the Helmet to be used with any CAPR face and headcover in any configuration with the 2061-08 and 2061-04A without need to change adapters, as was required on some previous configurations.

Although the adapters are held in place as they are snapped on to Helmet Snap Studs, additionally gluing them on, if desired, will ensure they stay attached to the Studs under anticipated usage.

Snap Adapters to the Helmet

Refer to Adapters and Configurations IFU P/N 03523254 for instructions for snapping helmet adapters on to the Helmet if they are not already secured on the Helmet.

Glue the Adapters to the Helmet



1. Place some Clear Gorilla Glue in the Syringe; 0.5 cc to 1.0 cc should be sufficient.

Secure the Syringe Tip on the Syringe.

Expel any air between the Gorilla Glue and the end of the Syringe Tip out of the Syringe.



2. Dispense enough Gorilla Glue into the Front adapter to completely cover the Snap Stud.



3. Draw off (siphon back into the Syringe) any excess glue. The glue only needs to completely cover the silver colored metal Snap Stud.

4. Repeat 1-3 for each side Adapter.



NOTE

Allow glue to cure for 24 hours before use.